


Remove the Neck Strap and Chin Pad from the Headmaster Collar.

1

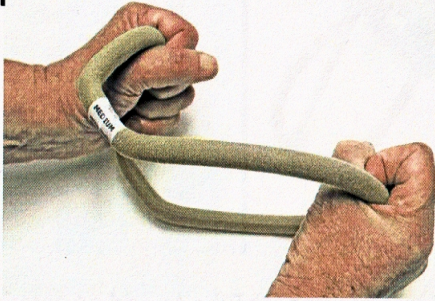

Bend Headmaster Collar to the desired height and fit according to directions enclosed with collar.

- Clip Top Tube on to the Collar

2




- Push the clip over the seam of the fabric
- Clip has a cut out at an angle for easier attachment

www.NeckSolutions.com

- Replace Chin Pad (included with the Anterior Support Tube)


3

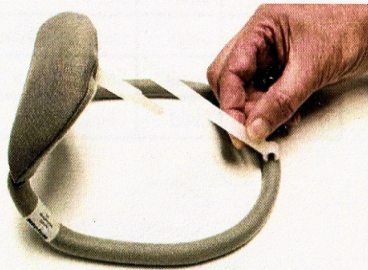

- Mark the desired length of the long tube and cut it to length

- Trim the fabric cover with scissors to cover the tubes
- Slide the fabric cover on to the long tube

4

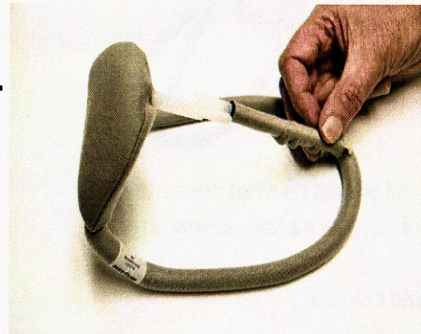

- Connect the short tube to the long tube

- Clip the lower clip to the Collar

5



Replace the Neck Strap

6



Note: The original Chin Pad may be used as a spare if you punch a hole underneath it.