


Attaching the Adapter to the Wheelchair

Attaching Wheelchair Clamps

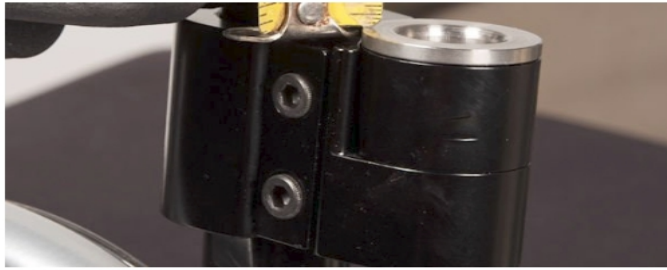

Place one Wheelchair Clamp at about the midpoint of the wheelchair cane. Lightly tighten the Clamp screws using the 4mm hex key.



Set the height of one Clamp, insert the Pin and swing the Cross Tube over to set the height of the other Clamp.



Depth Adjustment

Rotate the Wheelchair Clamps on the wheelchair canes to adjust the depth of the Back. The Cross Tubes will telescope. Tighten the Cross Tube adjustment screws to set the width.



Test the Pin release by releasing the levers and lifting the hardware off the wheelchair. Adjust the position of the Wheelchair Clamps if needed to ensure the Pins lift out easily.

The spacing of the screws is the industry standard and used by Stealth, Whitmyer etc. It will accept all of our headrest mount receivers.

For More Information, Please visit

www.NeckSolutions.com